3T Aerosol Collection Set

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
3T Aerosol Collection Set


These instructions describe the Installation, In-Process Checks, Emptying & Disposal of the 3T Aerosol Collection Set (050900100). For additional detail, please refer to the Heater-Cooler System 3T Operating Instructions. The aerosol collection set must be replaced within 7 days of its installation date. The aerosol collection set must also be replaced after the heater-cooler is disinfected.

1 Installation

1.1

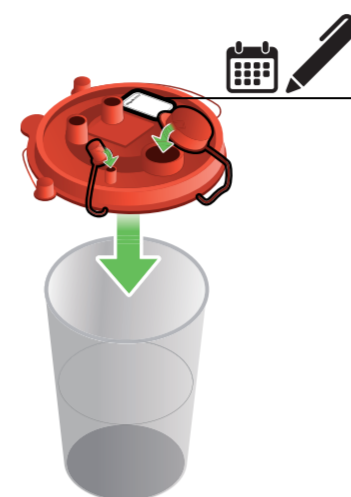
1.2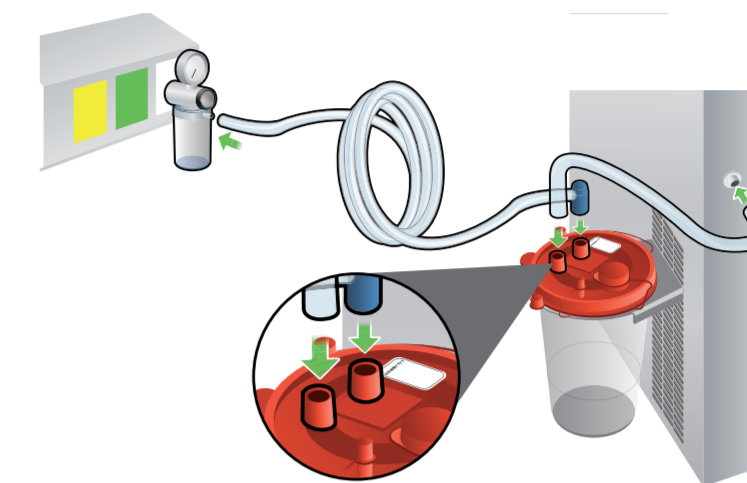


2 In-Process Checks


2.1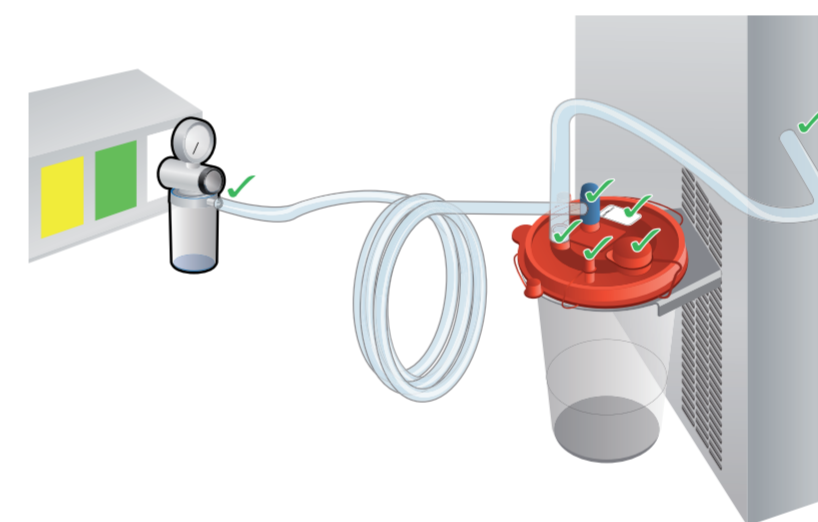

2.2

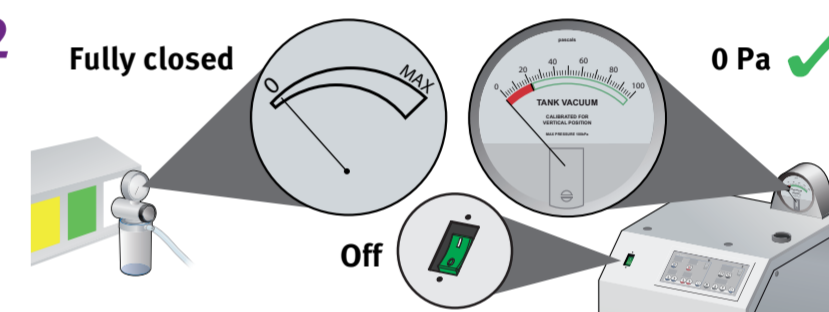
2.3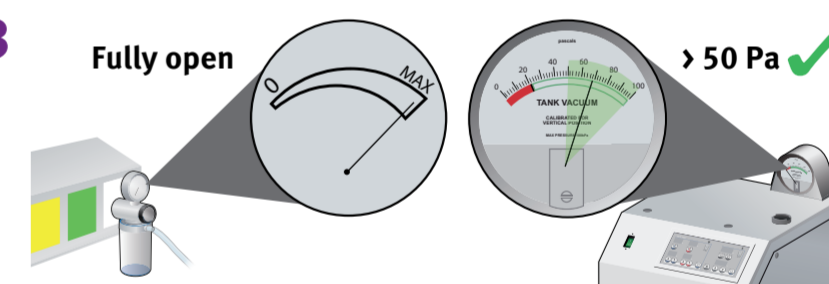


3 Emptying


3.1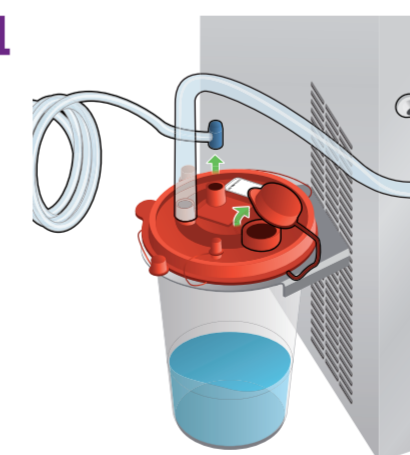

3.2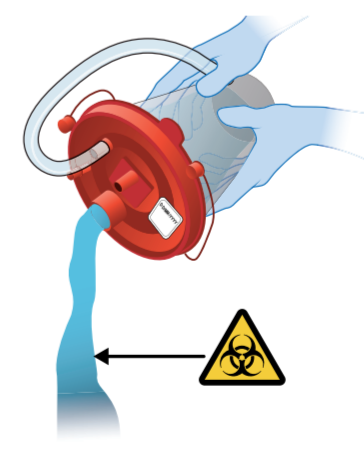

3.3

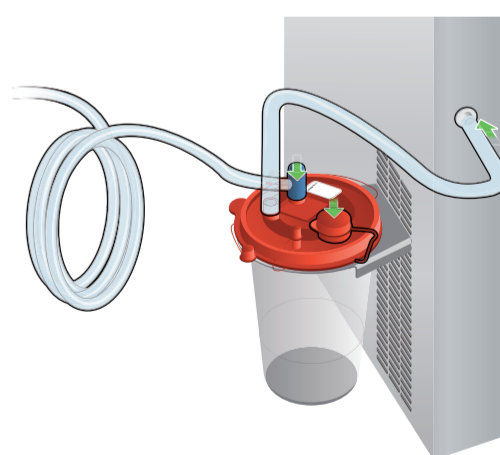4 Disposal

Dispose of the aerosol collection set according to your local environmental regulations and hospital policy.

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ENG – English

These instructions describe the Installation, In-Process Checks, Emptying & Disposal of the 3T Aerosol Collection Set (050900100). For additional detail, please refer to the Heater-Cooler System 3T Operating Instructions.

Important: The aerosol collection set must be replaced within 7 days of its installation date. The aerosol collection set must also be replaced after the heater-cooler is disinfected.

- Installation
- In-Process Checks
 - 2.2 With vacuum fully closed and heater-cooler off, heater-cooler pressure equals 0 Pascal
 - 2.3 With vacuum fully open, heater-cooler pressure is greater than 50 Pascal
- Empty the aerosol collection canister in accordance with your hospital’s policies.
- Dispose of the aerosol collection set according to your local environmental regulations and hospital policy.

 Producent	 LOT Batchkode
 Fremstillingsdato	 Holdbarhedsdato
 REF Katalognummer	 Bortskaffes inden for 7 kalenderdage
 i Se brugsanvisningen	 Biologisk fare
 Dato	

ENG – English

These instructions describe the Installation, In-Process Checks, Emptying & Disposal of the 3T Aerosol Collection Set (050900100). For additional detail, please refer to the Heater-Cooler System 3T Operating Instructions.

Important: The aerosol collection set must be replaced within 7 days of its installation date.

- The aerosol collection set must also be replaced after the heater-cooler is disinfected.
- Installation
 - In-Process Checks
 - 2.2 With vacuum fully closed and heater-cooler off, heater-cooler pressure equals 0 Pascal
 - 2.3 With vacuum fully open, heater-cooler pressure is greater than 50 Pascal
 - Empty the aerosol collection canister in accordance with your hospital’s policies.
 - Dispose of the aerosol collection set according to your local environmental regulations and hospital policy.

 Manufacturer	 LOT Batch code
 Te gebruiken vóór	 Use by
 REF Catalog number	 Dispose within 7 calendar days
 i Consult instructions for use	 Biological hazard
 Date	

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